

IMPORTANT: FLEXI CLEAN concentrated denture cleaner solution can be used effectively and safely for 7 days. The solution should be discarded and replaced for a new mix, repeat the Instructions for mixing every week.

Do not use bleach, tooth paste, or use any other abrasive solutions that may have chemical effects on the texture, color & shine of your Perfex dental appliance.

CAUTION: Do Not Put Solution Directly In The Mouth! Use Solution Only As Instructed .

Always Rinse Thoroughly. Keep Away From Children. Keep Away From Colored Fabrics.

*Not recommended for use on partials fabricated with Ticonium Alloy.

To order Perfex Flexi Clean please contact your dentist.

Each mini-jar contents approx.1 month supply 7Gr. (1/4oz)

All rights reserved to Perfex Ltd.

10 Smilansky St., Netanya, 42432, Israel,
Tel.+972 (0) 9-8825473, Fax:+972 (0) 9-8614474

www.perflexltd.com



How To Care For Your New Perfex Appliance

T-CRYSTAL
SEMI FLEXIBLE DENTURES

FLEXI NYLON
FLEXIBLE DENTURES

ACRY-FREE
FREE MONOMER
ACRYLIC DENTURES

ACETAL
DENTURES AND APPLIANCES



Perflex restorations, like your own teeth, require care and good oral hygiene. In order to maintain your new restoration in optimal condition, we recommend following the instructions below:

1. Continue brushing your teeth and gums, as instructed your dentist.
2. Hygienically cleanse your Perflex thermoplastic appliance by using **FLEXI CLEAN** concentrated denture cleaner.

FLEXI CLEAN Concentrated denture cleaner and stain remover is specially formulated to Kill ACTUAL germs and pathogenic microorganisms present on removable appliances. Eliminate Candida albicans Infections from thermoplastic dentures and acrylic dentures.

While disinfecting, PERFLEX FLEXI CLEAN will also remove coffee, tea, tobacco, fruit and iron stains, calcium, calculus, tartar and plaque in just 20 minutes.

This new formula non-chlorinated denture cleaner will not harm the denture base, partial denture, metal or soft reline material when used according to the Instructions.

INSTRUCTIONS FOR USE: 1 MINI-JAR

(1 month supply)

Mix one quarter mini-jar or 4 spoons powder with the enclosed spoon into 3/4 (200ml) cup of warm water. The solution may be used for 7 days before discarding. Rinse the dental appliance and place it in the solution for 20 minutes. Remove the appliance, brush loosened particles with a soft brush under running water before use. Use it daily. Discard and mix again every week.

INSTRUCTIONS FOR USE: FLEXI CLEAN KIT

(12 month supply)

Remove contents from "SOLUTION CONTAINER" and store on the box.

Fill the SOLUTION CONTAINER till the mark 200ml with warm to mild (not hot) water.

Mix 4 spoons powder with the enclosed spoon.

The solution may be used for 7 days before discarding. Rinse the dental appliance and place it in the solution for 20 minutes.

Remove the appliance, brush loosened particles with a soft brush under running water before use. Use it daily.

Close the "SOLUTION CONTAINER" after each use.

Discard and mix again, every week.

*A severely stained denture may require an overnight soaking (soft reline cases should be limited to 20 minutes)